**FERRI-Check® Self-test**

Ref. 25084BI

Self-diagnostic test for the detection of iron deficiency in whole blood

Anaemia due to iron depletion is widely held in children and women of all ages but mainly in women who still have their period (at least 20% suffer iron deficiency). Main signs as paleness, feeling tired, headaches, faster heartbeat or shortness of breath during exercise appear gradually and could go unnoticed. Therefore it is important to determine if the available iron is sufficient for the body's needs. Iron deficiency occurs when blood does not contain enough red cells and thus low levels of haemoglobin which is a major protein involved in oxygen transport in whole body. An important component of haemoglobin is iron.

Depletion in iron, which can happen during pregnancy, growth, in case of insufficient iron intake, inadequate absorption or blood loss (periods, abnormal bleedings, ulcers...etc) have tremendous effects on health.

**Caution: This test is not appropriated for patient suspected of or suffering from hemochromatosis.**

The FERRI-Check® is a rapid immunodiagnostic test for the assessment of ferritin (a protein capable of storing iron in cells) from finger prick whole blood sample. Therefore FERRI-Check® could be used for the screening of potential iron deficiency.

**PRESENTATION**

The box contains the material needed to perform a test:

- 1 sealed aluminium pouch containing:
    - 1 test device
    - 1 plastic pipette
    - 1 dessicant pouch
- Only open the protection pouch when you are ready to use the test. The desiccant pouch should not be used.
- 1 sterile lancet for blood sampling (+ one extra)
  - 1 diluent vial of 1 mL of diluent.
  - 1 instruction leaflet.

**MANUAL SELF-TEST**  
FERRI-Check® Self-test

quicktest.eu

**Result interpretation**

The intensity of the line colour does not have any importance for the interpretation of the test result.


**1. Positive result**

Only one coloured line appears under the mark C (Control). This result means that the ferritin concentration in blood is too low. Reserves insufficient. You should consult a doctor because you may suffer of iron deficiency.


**2. Negative result**

Two coloured strips appear in the window under the marks T (Test) and C (Control). The intensity of the line T may be clearer than the intensity of the line C. This result means that the ferritin concentration in blood is normal and that there is no potential iron deficiency.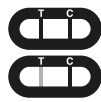

**3. Non valid result**

No line appears or a coloured line appears under the mark T (Test) without any line under the mark C. In this case, it is not possible to interpret the test, which must be considered as non valid. It is recommended to repeat the test with a new FERRI-Check® device.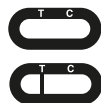

**Questions and answers**

**How does FERRI-Check® work?**

Ferritin is a protein and the primary form of iron stored inside cells. A positive result means that ferritin concentration in blood is lower than 20ng/mL and a possible iron deficiency.

**When can this test be performed?**

FERRI-Check® test must be performed in case of symptoms like paleness, feeling tired, headaches, faster heartbeat or shortness of breath during exercise mainly when pregnant or in case of copious periods. The test can be used at any time of the day but must not be performed in case of diseases, acute inflammations or in case of spleen or liver injury. Positive results could be obtained even in case of no iron deficiency situation.

**Can the result be incorrect?**

**The results are accurate as far as the instructions are carefully respected.** Nevertheless, the result can be incorrect if FERRI-Check® test gets wet before test performing or if the quantity of blood dispensed in the sample well is not sufficient. The plastic pipette provided in the box allows making sure the collected blood volume is correct.


**How to interpret the test if the colour and the intensity of the lines are different?**

The colour and intensity of the lines have no importance for result interpretation. The lines should only be homogeneous and clearly visible. The test should be considered as negative whatever the colour intensity of the test line.

**What is the line that appears under the mark C (Control) for?**


When this line appears, it only means that the test was performing well.

1. Sample well
2. Interpretation window
3. Pipette
4. Grey protective cap
5. Trigger button
6. Blue cap
7. White cap
8. Diluent vial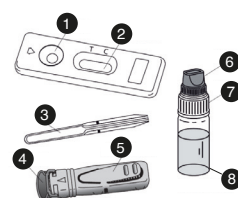

**Precautions**

1. External use only. DO NOT SWALLOW.
2. Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully respected.
3. Follow strictly the indicated blood and diluent quantities, as well as reading times.
4. Store between +4°C and +30°C. Do not freeze.
5. Do not use after the expiry date printed on the label and on the protective pouch or if this pouch is damaged.
6. Do not reuse the FERRI-Check® test.
7. Keep out of reach of children.
8. After use, all components can be discarded in a dustbin.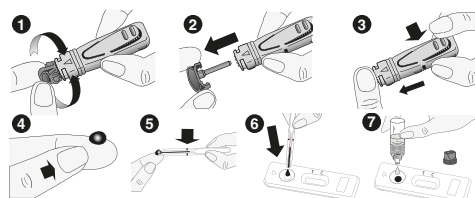

**Questions and answers**

**If I read the result after 15 minutes, will the result be reliable?**

No. The result should be read within 10 minutes after adding the diluent. The result is reliable up to 15 minutes.

**What do I have to do if the result is positive?**


If the result is positive, it means that the ferritin level in blood is lower than the norm (20ng/mL) and that you should consult a doctor to show the test result. Then, the doctor will decide whether additional analysis should be performed.

**What do I have to do if the result is negative?**

If the result is negative, it means that the ferritin level is higher than 20ng/mL and is within the norm. However, if the symptoms persist, it is recommended to consult a doctor.

**What is the accuracy of FERRI-Check®?**

The FERRI-Check® test is accurate and has been used for more than 10 years by professionals in the field. Evaluation reports show a result correlation of at least 98% with reference methods. Although this test is reliable, false positive or false negative results could be obtained.

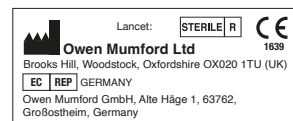	Read the instructions before use		For in vitro diagnostic use		Do not reuse
	Store between +4° and +30°C		Batch number		Expiry date
	Manufacturer	MD- 080020 m5 c - Revised in January 2022			

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**PROCEDURE**

1. Wash your hands thoroughly. Use soap and warm water. Dry your hand with clean towel.
2. Prepare the test device and the pipette. Take them out from the protective pouch (tear at the notch) and place them in the reach of your hands (you will need them later). Dispose the small dessicant bag.
3. Prepare the lancet. Hold the lancet **without touching the trigger button**. Unlock the lancet cap twisting it off ¼ turn until you feel it separates from the lancet and then continue twisting it (2-3 rotations). **Don't pull just twist** and discard the cap when finished (image 1 & 2).
4. Clean the end of the forefinger or of the middle finger with cotton damped with alcohol. Rub the chosen finger towards the tip to later on enhance the blood stream.
5. Press platform firmly against the previously cleaned finger, and press the release trigger button (image 3).
6. The tip will automatically retract into the body of the device.
7. Rub the finger's end to obtain enough whole blood sample (image 4).
8. Without pressing the bulb, put in contact the plastic pipette with the blood sample (image 5). The blood migrates into the pipette through capillarity **to the line indicated on the pipette**. You may rub again your finger to obtain more blood if the line is not reached. As far as possible, avoid air bubbles.
9. Put the blood collected with the pipette into the sample well of the device, by pressing on the pipette bulb (image 6).
10. Wait 30-40 sec for the blood being totally absorbed into the sample well. **Unscrew the blue cap** of the diluent vial (leave the white cap tightly screwed) and add the diluent as follows:
11. Hold the diluent vial vertically and slowly add **exactly 4 drops in the sample well** of the device (image 7) with an interval of 2-3 seconds between each drop.
12. Read the result after 10 minutes. Do not interpret after 15 minutes.

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