FSH-Check-1 Strip® Self-test
Ref. 11037BI

Self-diagnosis test for the determination of the follicle stimulating hormone in urine

SUMMARY AND EXPLANATION

The menopause is caused by a modification of the hormonal balance in woman body. This change arises generally when close to 45 years old. But true signs are observed around 55 years old. A menstrual cycle duration getting irregular is the early indication of the beginning menopause (perimenopause). The follicle stimulating hormone (FSH) is an hormone which increases during menopause. FSH-CHECK-1 STRIP can detect the presence of FSH in urine as soon as the concentration goes up from 25 mIU/mL and over.

KIT CONTENT

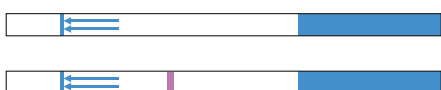
The kit contains two FSH-CHECK-1 STRIP tests and two urine containers along with one leaflet. Each strip is packaged in a sealed aluminium pouch containing one desiccant sachet. The strip should remain in the sealed pouch until use. Do not use the desiccant sachet.

Precautions

1. This test is designed for in vitro diagnostic use only. For external use only. DO NOT SWALLOW.
2. Carefully read the directions for use before performing the test. The test is reliable only if the directions for use are carefully respected.
3. Store between +4°C and +30°C. Do not freeze.
4. Do not use beyond expiration date (printed on the tag and on the protective pouch), and if the pouch is damaged.
5. Do not reuse FSH-CHECK-1 STRIP.
6. Keep out of reach of children.

Result interpretation

3. Invalid result
When no Control line develops, it is not possible to interpret the test, which must be considered as invalid. In this case, it is recommended to repeat the test with a new FSH-CHECK-1 STRIP and a fresh urine sample.

Questions and answers

How does FSH-CHECK-1 STRIP test work?

In women, the average hormonal cycle lasts 28 days. The FSH (Follicle Stimulating Hormone) secretion induces the follicle maturation, and LH (Luteinizing Hormone) secretion induces ovulation. The determination of FSH and LH level is indicated for the detection of amenorrhea and anovulatory cycles. Menopause arises at the mean age of 51 years old, i.e. between 45 and 55 years old depending on the female individuals.

Menopause is caused by a significant change in the balance of hormones. An irregular menstrual cycle duration is often the first sign of a beginning menopause. Other signs like hot flush, insomnia, palpitations, mouth ulcers, gingivitis, hair loss, dry vagina, osteoporosis, chills or excessive sweating could indicate menopause. The FSH-CHECK-1 STRIP test is a test for the determination of high levels of Follicle Stimulating Hormone in urine which is an indicator of menopause.

When the test could be used?
The test could be used at any time of the menstrual cycle.

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MANUAL SELF-TEST
FSH-Check-1 Strip® Self-test

quicktest.eu

Result interpretation

To interpret the results, it is necessary to compare the coloration of the FSH line (developing on the side of the blue arrows) with the coloration of the Control line (developing on the side of the strip grasp extremity).

1. Positive result

The result is positive if the FSH line is darker than the Control line. This result means that the FSH level is equal or higher than 25 mIU/mL. In this case, a second test must be performed 5 or 7 days later as the concentration of FSH can change during the menstrual cycle. If the positive result is confirmed, you could be under menopausal process and you are advised to consult your gynaecologist for a definitive diagnostic.

2. Negative result

The result is negative if the Control line is darker than the FSH line. This result means that the concentration of FSH is lower than 25 mIU/mL and that you are probably not under menopausal process.

Caution: in some cases, the FSH line may not develop at all.

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

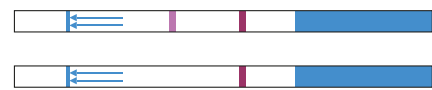
Directions for use

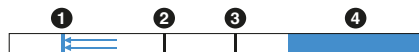1. Collect your urine in the urine container, included in this package (your urine is 1.2 cm high in the container).
 2. Tear up the protective pouch (notch) and only remove the strip from the pouch. Discard dessiccants.
 3. Vertically immerse the strip into the container with urine during 5 to 10 seconds. See illustration below.
- CAUTION: THE URINE QUANTITY SHOULD NOT PASS THE HORIZONTAL LINE THAT IS UNDER THE ARROWS (on the strip).**
4. Place the strip on a horizontal surface, and let the reaction develop.
 5. Read the result after 5 minutes. Do not interpret after 10 minutes.

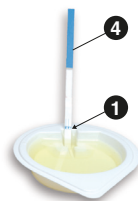1. Urine level indicator
2. FSH line
3. Control line
4. Grasp extremity

Questions and answers

What is the best moment to use the test?

You can perform the test at any moment of the day; however, the first urine in the morning contains a large quantity of FSH and allows the best results to be obtained.

Is it possible to get incorrect results?

The results are accurate as long as the directions for use are carefully followed. Nevertheless, the result can be incorrect if the FSH-CHECK-1 STRIP test is damp before performing the test. Moreover, the FSH-CHECK-1 STRIP test could be exceptionally distorted (positive result for a young woman). In case of unexpected or contradictory results, it is recommended to consult a gynaecologist, as some health problems can induce incorrect diagnosis.

I left the absorbent tip in urine for more than 10 seconds. Will I get an accurate result?

Yes. A few more seconds will not affect the results.

I left the absorbent tip in urine for less than 5 seconds. Will I get an accurate result?

No, because the sample volume might be insufficient.

What should I do if the result is positive?

If the result is positive with the first FSH-CHECK-1 STRIP test, a second test should be performed 5 or 7 days later. If a positive result is again obtained with a second test, the FSH level was elevated over an unusual period and could therefore indicate you are under menopausal process. You should consult your gynaecologist to confirm or not the diagnosis of menopause.

What should I do if the result is negative?

If the result is negative, the FSH level is lower than 25 mIU/mL, which means you are not probably under menopausal process. However, if you feel signs of menopause as indicated above, it is recommended to perform another test 40 to 60 days later or consult your gynaecologist.

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Would you like to see how it's done?
Scan the QR code with your mobile phone.

Questions and answers

Should the test be used as contraception method?

No, the test has not been designed for this purpose and should not be used for this purpose.

Will discontinuing my hormonal contraception (pill) affect the result?

If you use a contraceptive and that you have just discontinued it, it is possible that your cycles may be irregular for some time, which may affect the result. It is recommended to wait for two cycles before using the FSH-CHECK-1 STRIP tests.

I have reached the age to be menopausal. I have performed the test which result was positive but my menstrual cycle duration is still regular. Why?

A positive test indicates that the FSH level is equal or higher than 25 mIU/mL, which means that you are possibly under menopausal process. The menopause is defined as the cease of menstruation over a period of 12 months. However this biological change in a woman life is gradual and is preceded by a sooner or later period which is named peri-menopause. So the test result could be positive while your menstrual cycle duration is still regular. This means that you are in perimenopause phase.

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	Read instructions before use	IVD	For in vitro diagnostic use		Do not reuse
	Store between +4° and +30°C	LOT	Batch number		Expiry date
	Manufacturer				

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